

Strada Regina  
CH - 6874 Castel San Pietro - Switzerland  
Phone : + 41 91 696 60 60 Fax +41 91 696 60 66  
Info@medacta.ch – www.medacta.com

Manufactured by:  
Medacta International SA  
CH-6874 Castel San Pietro  
Switzerland

---

## **KNEE PROSTHESIS: MOTO™ Partial Knee System**

---

### **CAUTION**

Federal law (USA) restricts this device to sale, by or on the order of a physician.

### **ENGLISH – MOTO™ PARTIAL KNEE SYSTEM - INSTRUCTIONS FOR USE**

Important notice: the device(s) can be prescribed and implanted only by a doctor legally authorized to perform this type of surgery.

### **GENERAL**

Before any surgery, the surgeon must be familiar with the sales product literature and operative technique and must carefully read these instructions for use. Patient selection is as important as implant placement or positioning. Overweight patients, or unsuitable functional requirements may generate exceptional stresses and reduce the implant life. The warnings must be heeded, and the instructions for use must be strictly followed.

### **PRODUCT DESCRIPTION**

MOTO™ Partial Knee System, comprising individually packaged femur and tibial components, is designed for a single compartment replacement of the natural knee joint.

The femoral component is made of CoCr and the tibial component consists of a liner, made of ultra-high molecular weight polyethylene (UHMWPE), and a metal-back component made of Ti6Al4V.

### **INTENDED USE / INDICATIONS**

The MOTO™ Partial Knee System is designed for cemented use in partial knee arthroplasty, if there is evidence of sufficient sound bone to seat and support the components. Partial replacement of the articulating surfaces of the knee is indicated when only one compartment of the joint is affected due to the compartmental primary degenerative or post-traumatic degenerative disease, previous tibial condyle or plateau fractures, deformity or revision of previous arthroplasty.

### **CONTRAINDICATIONS**

Partial knee replacement is contraindicated in the following cases:

- Progressive local or systemic infection.
- Muscular loss, neuromuscular disease or vascular deficiency of the affected limb, making the operation unjustifiable.
- Osteoporosis or osteomalacia.
- Metabolic disorders which may impair bone formation.
- Rapid joint destruction, marked bone loss or bone resorption apparent on roentgenogram.
- Incomplete or deficient soft tissue surrounding the knee.
- Severe instability secondary to advanced destruction of condylar structures.
- Unicompartmental replacement is contraindicated in patients who have a permanent valgus or varus deformity that requires correction in order for the knee to function satisfactorily post-op.

Mental or neuromuscular disorders may create an unacceptable risk to the patient and can be a source of postoperative complications. It is the surgeon's responsibility to ensure that the patient has no known allergy to the materials used.

## **WARNINGS AND PRECAUTIONS**

The success of the operation depends on compliance with the operative technique supplied, and the proper use of the instrumentation supplied and specially designed for that range of implants. The trial instrumentation must be used to confirm the choice of sizes and verify the functionality of the joint.

The surgeon should implant this system only in patients with sufficient intact soft tissue and intact collateral and cruciate ligaments.

The MOTO™ Partial Knee System has not been evaluated for safety and compatibility in the MR environment. It has not been tested for heating, migration, or image artifact in the MR environment. The safety of the MOTO™ Partial Knee System in the MR environment is unknown. Scanning a patient who has this device may result in patient injury.

## **MEDACTA® INTERNATIONAL IMPLANTS**

Under no circumstances should a Medacta® International modular implant component be used in combination with a component from another manufacturer, unless otherwise specified by Medacta® International.

Only authorized Medacta® combinations should be used. Use MOTO™ Partial Knee System components only with other MOTO™ Partial Knee System components.

To determine whether these devices have been authorized for use in a proposed combination, please contact your Medacta® sales representative.

The components of a knee prosthesis should never be reimplanted. While an implant may appear undamaged, microscopic imperfections may occur and cause implant failure.

Always use a trial prosthesis for trial purposes only. Trial prostheses should not be assembled with components intended for permanent placement. Never adapt or alter trial prostheses. PE implants should be stored for at least three hours at 20°C (+/- 3°C) [62.6÷73.4°F] before the operation.

MOTO™ Partial Knee System prostheses are “non-pyrogenic”.

## **RISK FACTORS**

The following conditions, individually or together, may cause excessive loading of the affected limb, exposing the patient to greater risk of a partial knee arthroplasty failure:

- Obesity or overweight of the patient
- Hard physical labour
- Intense sporting activity
- High level of activity
- Probability of falling
- Alcoholism or drug addiction
- Other handicaps which could compromise the outcome of the operation.

The following conditions, individually or together, will make fixation of the knee prosthesis challenging:

- Advanced osteoporosis or insufficient bone stock
- Metabolic disorders or systemic medications leading to gradual loss of bone support for the prosthesis (e.g. diabetes mellitus, treatment by steroids, immunosuppressives, etc.)
- History of disseminated systemic or local infection
- Significant deformations preventing correct fixation or placement of the prosthesis
- Tumors of the supporting bone structures
- Allergic reactions to the prosthesis materials or cement
- Tissue reaction to implant corrosion or wear debris
- Functional incapacity of the other joints.

## **INSTRUCTIONS FOR USE**

### **PREOPERATIVE PHASE**

The surgeon should verify possible patient physical limitations and mental deficiencies and should also discuss with the patient all the details of the procedure and prosthesis. The discussion should consider the limitations of the procedure and the constraints imposed by the selected implant. The factors which could limit the performance and stability of the implant, e.g. level of activity, patient's weight, should be set out to improve the patient's chances to avoid complications.

The necessity to follow the postoperative instructions given by the surgeon should be fully understood by the patient.

A stock of sterile implants of suitable sizes should be available and checked before surgery.

The prerequisites for partial knee replacement include:

- Osteoarthritic damage to only one femorotibial or femoropatellar compartment of the knee joint
- Unicompartmental disease with a correctable deformity with minimal to no ligament releases

- Stability of collateral and cruciate ligaments
- Integrity of the quadriceps and popliteal tendon

### **HANDLING**

To avoid scratching or damaging the implants, the implants should be handled with the utmost care by qualified personnel and in an environment where conditions of hygiene are controlled. The implants should be kept in their undamaged packages until needed for use.

Do not use implants from opened packages, that are damaged, or that are beyond their expiration date.

### **SURGICAL TECHNIQUE**

The surgeon should be fully familiar with the surgical technique. Supplementary information about the surgical techniques and products are available on request. Careful preoperative planning, documented by X-rays, is essential. MOTO™ Partial Knee System specific templates are available.

### **POSTOPERATIVE CARE AND FOLLOW-UP**

The surgeon should caution the patients to control their level of activity and avoid excessive loads on the replaced joint, and make them aware of the precautions to be taken with regards to exercise, treatments and limitations on activities, as well as avoiding exposure to magnetic fields.

Periodic follow-up and X-rays are recommended to make comparisons with the immediate postoperative condition and identify implant displacement, loosening, etc. Excessive physical activity, and operated limb traumas may cause early failure of the arthroplasty through implant displacement, fracture and/or wear. If the case occurs, it is necessary to place the patient under supervision, evaluate the possible progression of the deterioration, and weigh the benefit of early revision.

### **ADVERSE EFFECTS AND COMPLICATIONS**

#### **GENERAL**

- Prosthesis dislocation, often related to the above-mentioned risk factors.
- Early or late loosening, tibial subsidence, bending, fissure fracture, fracture, deformation or wear of one or more of the prosthetic components, often related to the above-mentioned risk factors. Loosening can also occur as a result of an incorrect fixation or positioning of the components.
- Early or late infection which may require removal of the implant followed by arthrodesis or 2-stage reimplantation.
- Pain, dislocation, subluxation, flexion contracture, mobility reduction, leg shortening or lengthening, resulting from improper positioning, loosening or wear of the components.
- Excessive wear of the polyethylene component due to damage to the femoral component, loose cement or bone fragments and/or high levels of activity or weight.
- Fracture of the tibia or femur. Intra-operative fractures are usually associated with severe deformations and /or osteoporosis. Postoperative fractures are generally traumatic or fatigue fractures. They may result from cortex defects, multiple pin holes, former screw holes, misdirected reaming and/or uneven distribution of bone cement.
- Cardiovascular disorders and thromboembolic diseases, including thrombosis, embolism, and myocardial infarction.
- Tissue reactions, osteolysis and/or implant loosening caused by metal corrosion, allergy, wear debris, or loose cement particles.
- Myositis ossificans, especially in osteoarthritic males having a limited range of motion before the operation and/or a previous myositis. The incidence of myositis ossificans increases with past surgical history and in case of infection.

#### **INTRA-OPERATIVE**

Under no circumstances should the components come into contact with hard objects. Before use, each component must be visually inspected for imperfections. Special surgical instruments are required for knee surgery. It is important to review the use and handling of these instruments based on the surgical technique. The alignment and cutting templates must be inspected visually before operation. Distorted or damaged instruments may result in improper positioning of the prosthesis and arthroplasty failure. Careful cleaning and correct preparation of the bone surfaces are essential for the fixation of the prosthesis. Bone resection must be kept minimal. Excessive bone resection or excessive use of pins to secure the instruments may induce mechanical problems and bone resorption resulting in failure of the surgery. When preparing the bone surfaces and placing the components, it is necessary to check the components for correct alignment. Before closing the wound, the surgical site must be cleaned free of bone particles, residual cement and any foreign particles that may cause excessive wear. The range of motion and the level of constraint must be carefully checked and corrected, if necessary, to avoid incorrect seating, instability or encroachment.

### **IMMEDIATE POSTOPERATIVE**

- Cardiovascular disorders, including vein thrombosis, embolism, and myocardial infarction.
- Hematoma and/or delayed healing.
- Pneumonia and/or atelectasis.
- Subluxation or dislocation.
- Uncontrolled varus or valgus.

### **LATE POSTOPERATIVE**

- Poor range of motion due to incorrect selection or positioning
- Aggravation of the problems with the knee and ankle of the ipsilateral or contralateral limb caused by difference in leg length, femur displacement and/or muscular deficiency.
- Fracture of the patella resulting from excessive stress or intraoperative weakening.
- Bone resorption which may damage the fixation or result in implant loosening.
- Periarticular calcification or ossification which may reduce mobility and the articular range of motion, and or encroachment.
- Subluxation or dislocation.

### **PACKAGING**

All the implant components of the MOTO™ Partial Knee System prosthesis are supplied in single-use individual packages.

For components delivered sterile, the sterilization method is indicated on the label. The expiration date and package integrity must be checked to ensure that sterility of the contents has not been compromised. If the package is damaged, do not use the component. Do not resterilize.













### **INSTRUMENTS**

Instruments are supplied non-sterile and must be cleaned and sterilized prior to use. Recommended cleaning, decontamination and sterilization instructions are provided on [www.medacta.com](http://www.medacta.com)

### **STORAGE**

The packages must be stored in a cool, dry place, away from light.

### **SYMBOLS**

	Do not reuse
	Do not resterilize
	Caution, read the accompanying documents
	Consult instructions for use
	Do not expose to sunlight
	Store in a dry place
	Do not use if package is damaged
	Use by
	Lot number
	Reference number
	Sterilized with ethylene oxide
	Sterilized by irradiation

Date: January 2017

Distributed by Medacta USA, Inc. 3973 Delp St., Memphis, TN 38118