


# Medacta-C

MECTA-C CERVICAL INTERVERTEBRAL BODY FUSION DEVICE - UPGRADE

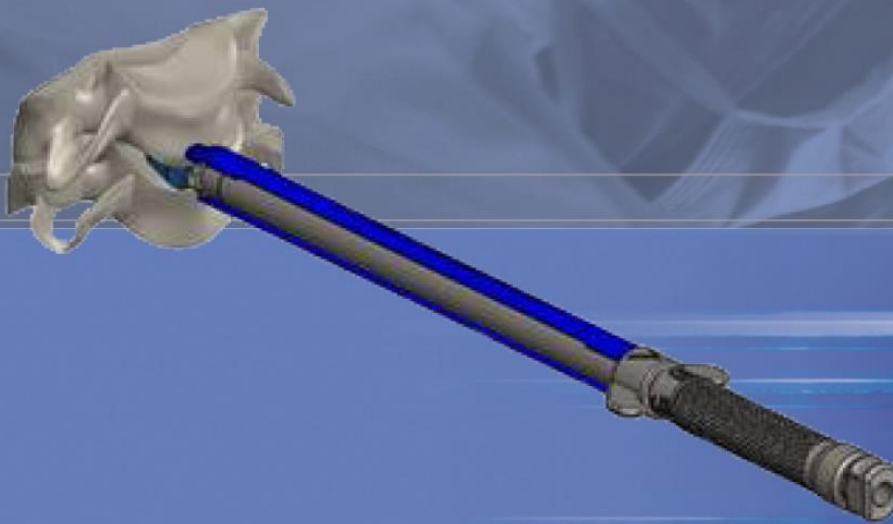

## Surgical Technique

Hip

Knee

Spine

Navigation

## INDEX

---

1	INTRODUCTION	3
2	IMPLANT POSITIONING	3

## 1 INTRODUCTION


The current document describes the surgical steps related to the Mecta C cage system upgrade

## 2 IMPLANT POSITIONING

Insert the bone graft into the device.  
Ensure that the Inserter thumb wheel is in the unlocked position.




Attach the implant to the Inserter.  
Turn the thumb wheel 90° to fix the implant firmly to the Inserter.

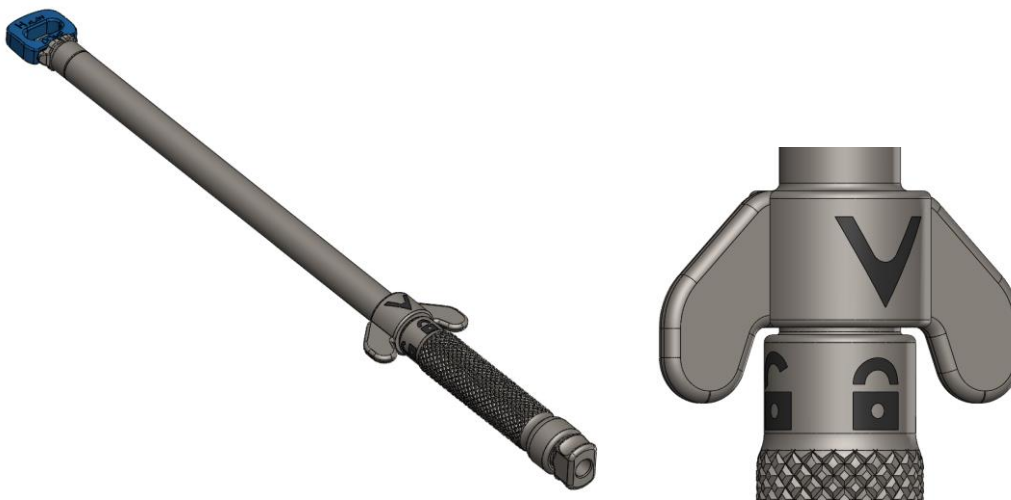


The lock symbol on the thumb wheel has to be aligned with the mark on the handle

Insert the implant straight into the intervertebral disc space by gentle impaction as described in technique 99.44c.12 (99.44c.12US, 99.44c.82).

The Depth Limitator can be coupled to outer shaft of the Mecta C Cage Inserter to help with the insertion of the cervical cage.

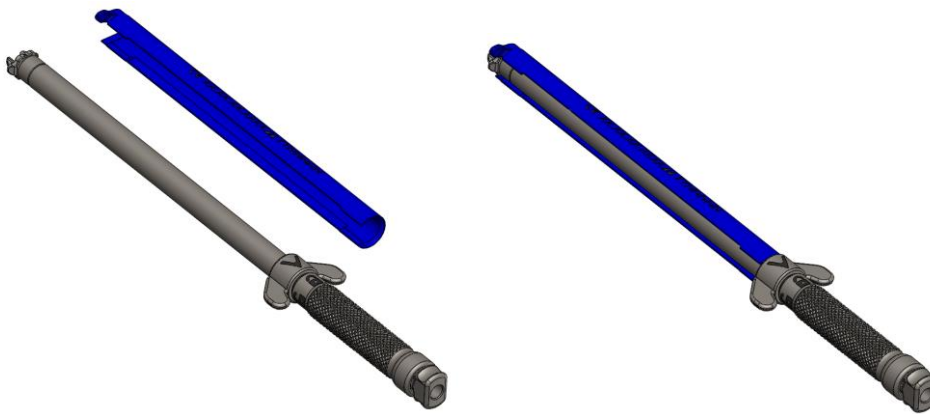

The Depth Limitator has a distal cranial and caudal struts that will contact the anterior edge of the vertebral body to provide a positive stop.

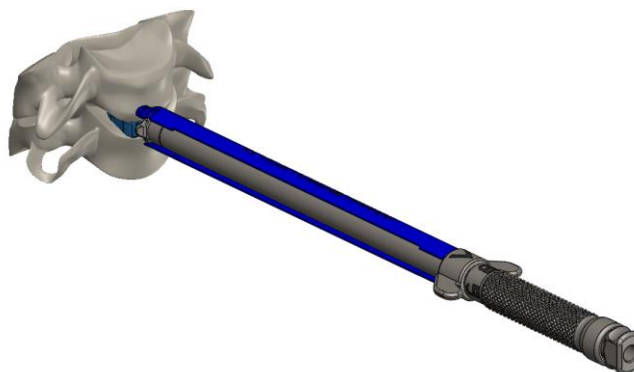

All the comprehensive steps of cervical preparation and cage insertion as well as cage final positioning check are detailed in the surgical technique 99.44c.12 (99.44c.12US, 99.44c.82) as reference.









International



## HEADQUARTERS

**Medacta International SA**  
Strada Regina - 6874 Castel San Pietro  
Switzerland  
Phone +41 91 696 60 60  
Fax +41 91 696 60 66  
info@medacta.ch

## REPRESENTATIVES

**Switzerland - Frauenfeld**  
Gewerbestrasse 3 - 8500 Frauenfeld  
Phone +41 (0) 848 423 423  
Fax +41 (0) 848 423 424  
info@medacta-swiss.ch

## SUBSIDIARIES

**Australia** Medacta Australia PTY.LTD - Unit A1, 16 Mars Road - Lane Cove - NSW 2066  
Phone +61 (2) 94202944 - Fax +61 (2) 94202578 - info@medacta.com.au

**Belgium** Medacta Belgium B.V.B.A./S.P.R.L. - 5a Rue de la Maîtrise - 1400 Nivelles  
Phone +32 (0) 67 555 482 - Fax +32 (0) 67 555 483 - info@medacta.be

**Canada** Medacta Canada Inc. - 31 McBrine Drive, Unit 11- N2R 1J1 - Kitchener, Ontario  
Phone +1 519 279 1934 - Fax +1 519 279 1938 - info@medacta.ca

**China** Medacta China - Room B, 32/F, New SH Intl Tower - No. 360 Pudong South Road - Shanghai 200120, China  
Phone +86 21 5835 1149 - info@medacta.cn

**France** Medacta France SAS - 6 Rue du Commandant d'Estienne d'Orves - Parc des Chanteraines - 92390 Villeneuve - La Garenne  
Phone +33 147 39 07 22 - Fax +33 147 39 73 17 - info@medacta.fr

**Germany** Medacta Ortho GmbH - Jahnstrasse 86 - D - 73037 Göppingen  
Phone +49 (0) 7161 50 44 30 - Fax +49 (0) 7161 50 44 320 - info@medacta.com

**Italy** Medacta Italia Srl - Via G. Stephenson, 94 - 20157 Milano  
Phone +39 02 390 181 - Fax +39 02 390 00 704 - mail@medacta.it

**Japan** Medacta Japan CO. LTD - 100-0014 Chiyoda House 201 - 2 - 17-8, Nagatacho, Chiyoda-ku, Tokyo  
Phone +81 (0) 3 5510 8883 - Fax +81 (0) 3 5510 8884 - info@medacta.co.jp

**Spain** Medacta España SLU - Avda de las Jacarandas - 2 - Edificio CREA Oficina 631 - 46100 - Burjassot  
Phone +34 (0) 963 484 688 - Fax +34 (0) 963 484 688 - info@medacta.es

**UK** Medacta UK Limited - 16 Greenfields Business Park - Wheatfield Way - Hinckley - Leicestershire - LE10 1BB  
Phone +44 (0) 1455 613026 - Fax +44 (0) 1455 611446 - info@medacta.co.uk

**USA** Medacta USA, Inc. - 1556 West Carroll Avenue - Chicago - IL 60607  
Phone +1 312 878 2381 - Fax +1 312 546 6881 - info@medacta.us.com

## DISTRIBUTORS

Argentina  
Indonesia

Austria  
Kuwait

Belarus  
Malaysia

Brazil  
Mexico

Bulgaria  
New Zealand

Colombia  
South Africa

Greece  
Vietnam

Mecta-C  
Surgical Technique

Ref: 99.44c.12addendum  
Rev.00

Last update: June 2016