

Instructions for use Twin Hose to the Woodpecker

- Power

The Woodpecker is a pneumatically powered device and must be operated at a pressure of 100-120 PSI (7-8 bar). Use nitrogen or clean, filtered compressed air to power this device.

- Air Hose

The Woodpecker is operated with a twin-hose (inner-hose \approx 18m and air outlet hose).


Bent or kinked hoses may prevent the unit from operating satisfactorily.



1. Quick disconnect fitting (on the left) to the Woodpecker (female)
2. Connection (on the right) to air supply (male)

Instructions for use (cont.)

- Pull hose quick-disconnect outer-shell back and attach the hose to the Woodpecker

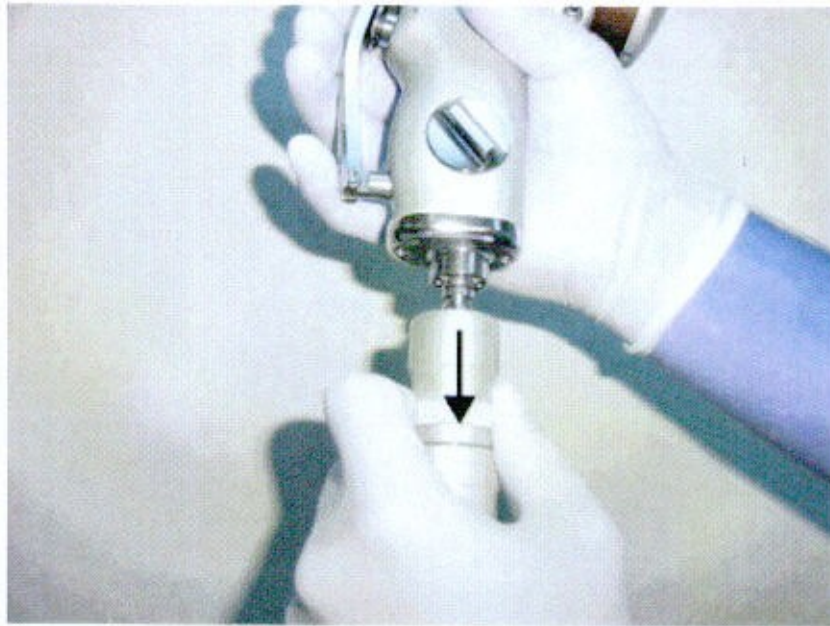

- Slide quick-disconnect outer-shell in direction shown to secure hose to the Woodpecker.

