

# Medacta<sup>®</sup>Flip

INTRA ARTICULAR EXPANDER



## CREATE THE SPACE YOU ALWAYS WANTED

### FIRST AND ONLY DISPOSABLE INTRA ARTICULAR EXPANDER

Designed to retract the hip joint capsule

### IMPROVED VISIBILITY

In the intra-operative space


### UTILIZE MULTIPLE DEVICES AT THE SAME TIME

When more visibility is required

### TECHNICAL DATA

Available in 3 sizes

Working Length: 80mm / 120mm / 150mm

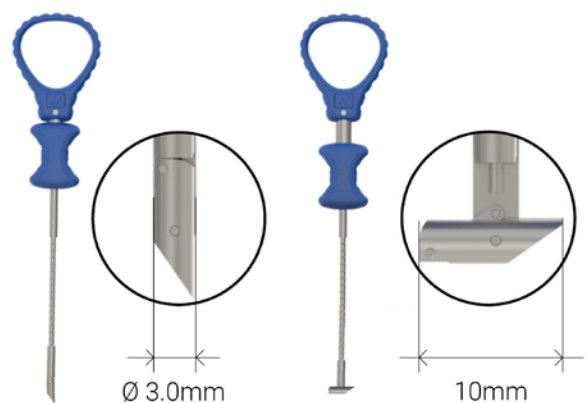


## SURGICAL STEPS




**1**  
INSERT THE MAIN BODY OF THE MECTAFLIP through the skin until the desired position is reached.



**2**  
FLIP THE TIP by moving the Plastic Barrel.

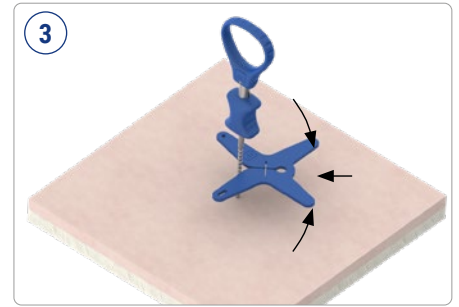

**3**  
COUPLE THE STAR COUNTER PLATE over the main body.

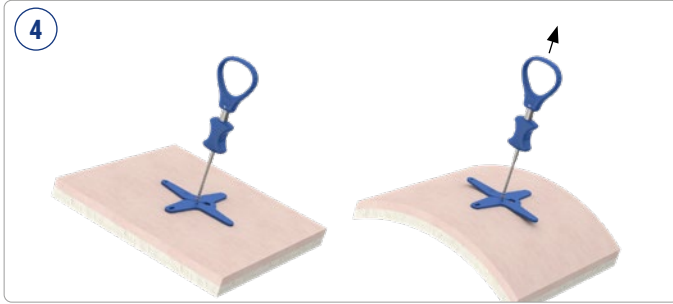

**4**  
PRESS THE COUNTER PLATE TOWARDS THE SKIN while pulling the Ring Handle with the other hand to compress and raise the tissue.



**5**  
TIP: Multiple devices can be used at the same time if more visibility is required.

## ORDERING INFORMATION

Order Number	Description	Order Number	Description
05.09.10.0001	MectaFlip 80mm	05.09.10.0003	MectaFlip 150mm
05.09.10.0002	MectaFlip 120mm		

## RELATED PRODUCTS

Order Number	Description	Order Number	Description
05.11.10.0041	Hip Access Kit	05.14.10.0007	Hip cannula Ø8.5x90mm
05.11.10.0040	Blade Handle	05.14.10.0008	Hip cannula Ø8.5x120mm
05.11.10.0050	FastShuttle CC - Suture Passer Capsular Closure	05.14.10.0010	Hip Metal Trocar Ø6.1x90mm
05.11.10.0051	FastShuttle CC - Disposable Needle STERILE (5ea)	05.14.10.0011	Hip Metal Trocar Ø6.1x120mm
05.14.10.0022	Hip Cannula System Ø8.5x90mm	05.14.10.0015	Hip Metal Trocar Ø8.5x90mm
05.14.10.0023	Hip Cannula System Ø8.5x120mm	05.14.10.0016	Hip Metal Trocar Ø8.5x90mm
05.14.10.0005	Hip cannula Ø6.1x90mm	05.14.10.0030	Hip Obturator - 90mm
05.14.10.0006	Hip cannula Ø6.1x120mm	05.14.10.0031	Hip Obturator - 120mm





[SPORTSMED.MEDACTA.COM](https://SPORTSMED.MEDACTA.COM)


Distributed by **Medacta International SA** | Strada Regina - 6874 Castel San Pietro - Switzerland | Phone +41 91 696 60 60 | Fax +41 91 696 60 66 | [info@medacta.ch](mailto:info@medacta.ch)

Find your local dealer at: [medacta.com/locations](https://medacta.com/locations). All trademarks and registered trademarks are the property of their respective owners. This document is not intended for the US market. Please verify approval of the devices described in this document with your local Medacta representative.

ref: 99.111SMH.1SALES-S rev. 00 Last update: May 2022