

# MectactaQTH<sup>®</sup>

QUAD TENDON HARVESTING



## MINIMALLY INVASIVE QUAD TENDON HARVESTING

**PREDICTABLE CROSS-SECTION**

**REDUCED HARVEST SITE MORBIDITY**

**SAFE TENDON TRANSPORTATION**

**DEDICATED M-ARS ACL CUTTERS**

**LOCKING MECHANISM TO PREVENT  
PREMATURE TENDON TRANSECTION**

### WIDE PRODUCT RANGE


- 13 Vertical & 7 Horizontal Blades (disposable)
- 2 Subcutaneous Tendon Cutters (small & large)
- 1 MectactaQTH Cutter Handle (reusable)




# SURGICAL STEPS




With knee in 90° flexion, create a 2.5 - 3 cm transverse incision over the superior pole of the patella.

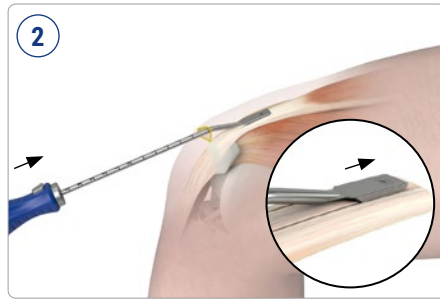


Select the size of the Vertical Cutter according to the desired width of the harvested graft. Using the dedicated handle, insert and advance the instrument to achieve the desired graft length.

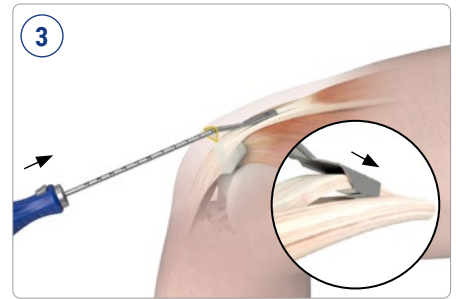


Select the size of the Horizontal Cutter according to the desired thickness of the harvested graft. Using the dedicated handle, insert and advance the instrument across the desired graft length.

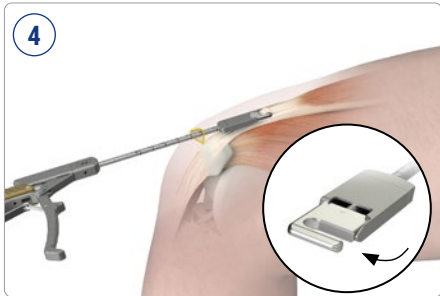

With the Subcutaneous Cutter fully open, insert the graft segment into the dedicated tendon slot.

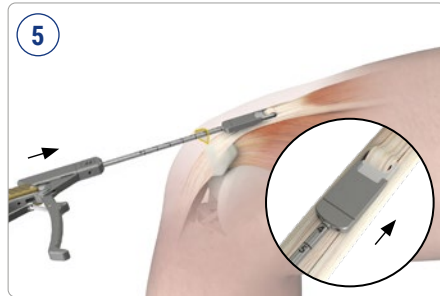

Making sure that the locking mechanism is enabled, depress the Subcutaneous Cutter handle to lock tendon within the slot and advance the instrument to desired graft length.

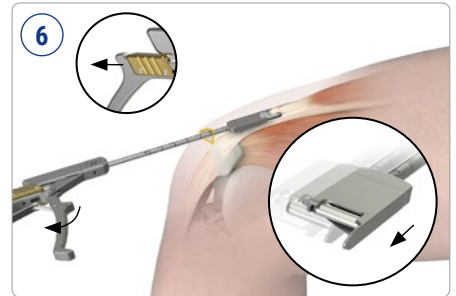

Once at the desired graft length, retract the locking mechanism slider to free the handle and perform the proximal cut.

# ORDERING INFORMATION

VERTICAL CUTTERS (DISPOSABLE)			
Order Number	Description	Order Number	Description
05.05.10.0094	Vertical Cutter - 8x5	05.05.10.0101	Vertical Cutter - 12x7
05.05.10.0095	Vertical Cutter - 8x7	05.05.10.0102	Vertical Cutter - 14x5
05.05.10.0096	Vertical Cutter - 9x5	05.05.10.0103	Vertical Cutter - 14x7
05.05.10.0097	Vertical Cutter - 9x7	05.05.10.0104	Vertical Cutter - MARS ACL SMALL
05.05.10.0098	Vertical Cutter - 10x5	05.05.10.0105	Vertical Cutter - MARS ACL MEDIUM
05.05.10.0099	Vertical Cutter - 10x7	05.05.10.0106	Vertical Cutter - MARS ACL LARGE
05.05.10.0100	Vertical Cutter - 12x5		

HORIZONTAL CUTTERS (DISPOSABLE)			
Order Number	Description	Order Number	Description
05.05.10.0107	Horizontal Cutter - 10x4	05.05.10.0111	Vertical Cutter - MARS ACL SMALL
05.05.10.0108	Horizontal Cutter - 15x3	05.05.10.0112	Vertical Cutter - MARS ACL MEDIUM
05.05.10.0109	Horizontal Cutter - 15x5	05.05.10.0113	Vertical Cutter - MARS ACL LARGE
05.05.10.0110	Horizontal Cutter - 15x6		

DEDICATED INSTRUMENTS	
Order Number	Description
05.05.10.0092	Subcutaneous Cutter Small
05.05.10.0114	Subcutaneous Cutter Large
05.05.10.0093	MectaQTH Cutter Handle



SPORTSMED.MEDACTA.COM |

Distributed by Medacta International SA | Strada Regina - 6874 Castel San Pietro - Switzerland | Phone +41 91 696 60 60 | Fax +41 91 696 60 66 | info@medacta.ch

Find your local dealer at: medacta.com/locations. All trademarks and registered trademarks are the property of their respective owners. This document is intended for the US market. Please verify approval of the devices described in this document with your local Medacta representative.

ref: 99.107SMK.1SALESSUS rev. 00 Last update: December 2021