

SHOULDER GENERAL INSTRUMENTS



Surgical Technique

Joint

Spine

Sports Med

INDEX

1. INSTRUMENTS OVERVIEW	4
1.1 Probe (Ref. 05.10.10.0001)	4
1.2 Elevator (Ref. 05.10.10.0002)	4
1.3 Rasp (Ref. 05.10.10.0003)	4
1.4 Knot pusher (Ref. 05.10.10.0004)	4
1.5 V-Pipe	5
2. INSTRUMENTS TRAY	6

1. INSTRUMENTS OVERVIEW

Medacta's aim is to provide the surgeons a complete set of instruments that could be used during the arthroscopic treatment of the most common shoulder pathologies: torn or damaged labrum or ligaments, joint instability, torn tendons, loose bodies removal, arthroscopic treatment of inflammation phenomena, etc.

1.1 PROBE (Ref. 05.10.10.0001)

Designed to palpate soft tissues or the intra-articular structures during the arthroscopic procedures. It is provided with a 3mm tip with laser marking every 2mm on the tip and every 10mm along the shaft.

Tip 3 mm, straight shaft, lasermarking measures, WL 150 mm



1.2 ELEVATOR (Ref. 05.10.10.0002)

It is provided of a thin wedged tip to raise the tissues from the bone surface. It is designed with a bent tip (20°).

Bent thin tip 20°, Straight shaft, WL 150mm



1.3 RASP (Ref. 05.10.10.0003)

It is designed to rasp the bone surface during arthroscopic procedures. It is provided of a teathed tip bent upwards (20°).


Rasp tip bent upwards (20°), WL 150mm



1.4 KNOT PUSHER (Ref. 05.10.10.0004)

Designed with a spherical internal cavity to better accommodate the suture knot together with an atraumatic blunt tip.

Atraumatic blunt tip, spheric internal cavity, WL 150mm

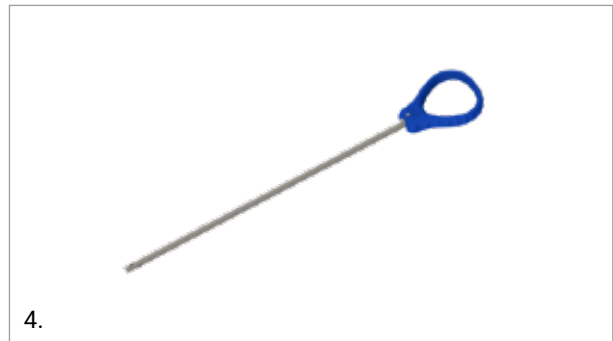

1.5 V-PIPE

Provided of atraumatic tip to avoid damage to soft tissues during the insertion into the arthroscopic portals. It is also designed with a V-Shape trail to better guide the tip of the instruments inside the portals. Available with rounded and reduced tips for easy insertion inside the portals. Combined with a dedicated detachable ergonomic handle with locking button.



V-Pipe detachable handle (Ref. 05.10.10.0007)

Ergonomic handle with locking button



V-Pipe Rounded tip (Ref. 05.10.10.0005)

Rounded atraumatic tip, internal V-shape, connection with detachable handle, WL 90mm



V-Pipe Reduced tip (Ref. 05.10.10.0006)

Reduced atraumatic tip, internal V-shape, connection with detachable handle, WL 90mm



2. INSTRUMENTS TRAY

Optional metal tray designed with dedicated brackets to contain all the instruments of the set.

References included (1 level tray)

SPORTS MEDICINE - SHOULDER GENERAL SET (Ref. 05.10S.001)



Ref.	Description	Qty.
05.10.10.0001	Probe	1
05.10.10.0002	Elevator	1
05.10.10.0003	Rasp	1
05.10.10.0004	Knot pusher	1
05.10.10.0005	V-Pipe rounded tip	1
05.10.10.0006	V-Pipe reduced tip	1
05.10.10.0007	V-Pipe handle	1

Part numbers subject to change.

NOTE FOR STERILISATION

The instruments are not sterile upon delivery. It must be cleaned before use and sterilised in an autoclave noting the regulations of the country, US directives where applicable and following the instructions for use of the autoclave manufacturer. For detailed instructions please refer to the document "Recommendations for cleaning decontamination and sterilization of Medacta International orthopaedic devices" available at www.medacta.com.



**REDEFINING BETTER
IN ORTHOPAEDICS
AND SPINE SURGERY**

MEDACTA.COM



Medacta International SA
Strada Regina - 6874 Castel San Pietro - Switzerland
Phone +41 91 696 60 60 - Fax +41 91 696 60 66
info@medacta.ch

Find your local dealer at: medacta.com/locations

All trademarks and registered trademarks are the property of their respective owners.
This document is intended for the US market.

Shoulder General Instruments
Product Catalogue

ref: 99.106SMS.180US
rev. 00

Last update: April 2019